

S3 CART/TABLETOP KIT

ASSEMBLY INSTRUCTIONS




QTY.	DESCRIPTION	PART NO.
1. (1x)	Antenna	32-0020-A
2. (1x)	Power Cord	32-PC-1880
3. (1x)	Barcode Scanner	39-6001
4. (2x)	Rubber Bumper	40-1746
5. (2x)	Pan Head Screw 6-32 x 1 1/2	41-PHMS-6-55
6. (4x)	Sems Pan Head Screw M4.7x10	41-SEMS-M4-10-ND
7. (1x)	Monitor	69-0096
8. (1x)	Power Supply Outlet	30-1558-1
9. (1x)	Screwdriver, ratchet stub	99-5729-4
10. (2x)	Mount Rail	T19835
11. (4x)	Sems Pan Head 8-32 x 3/8	41-sems-8-20


1) Remove the monitor from the box. You can discard all other items in the box.



2) Remove the stand cover by pushing the two clips in and pulling out. Keep to the side for now.

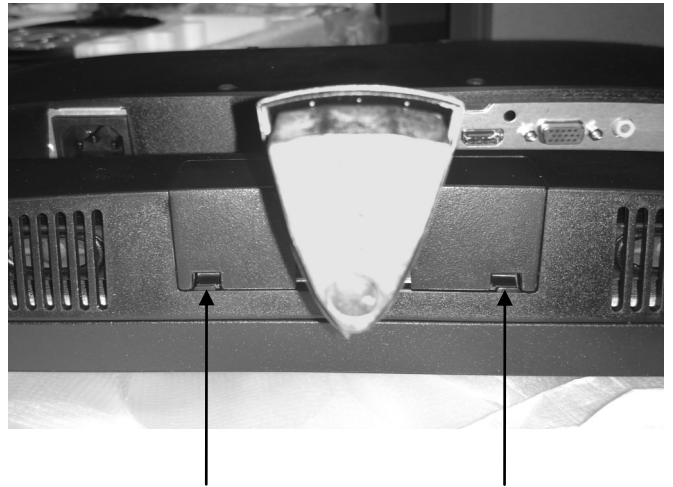

3) Remove the two screws on the stand.

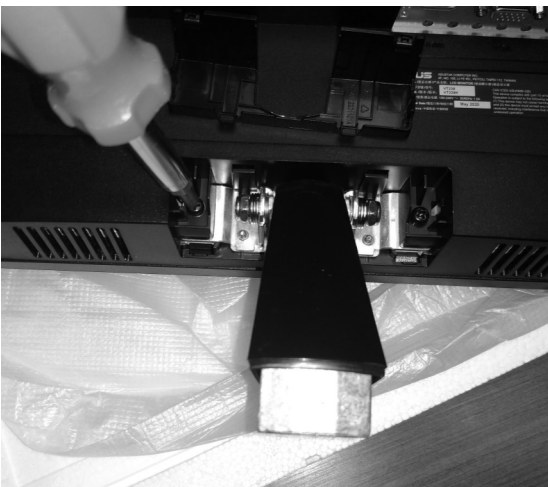


4) Slide the stand out.



5) Replace the stand cover.



6) Remove the 4 Vesa plugs on the back of the monitor. These can easily be popped out with your finger. (Removed plugs shown below)

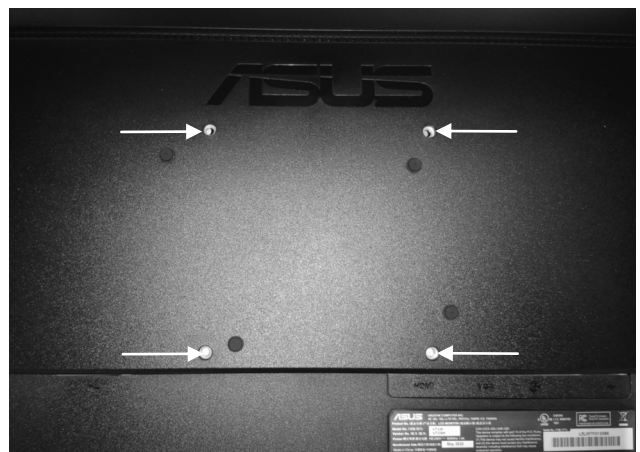

7) Make sure the top of the monitor stand is laying flat.





8) Line up the 4 holes on the back of the monitor with the monitor stand.




9) Using the 4 Sems Pan Head screws and the provided screwdriver, secure the monitor to the monitor stand.



10) Locate the 3 ports on the bottom of the monitor and plug in the appropriate cables.

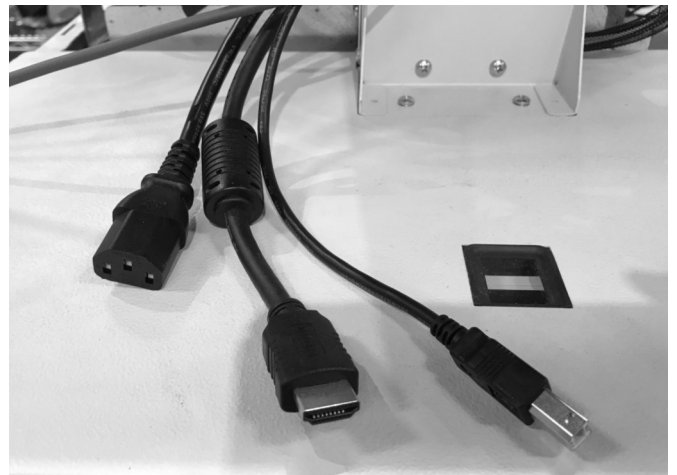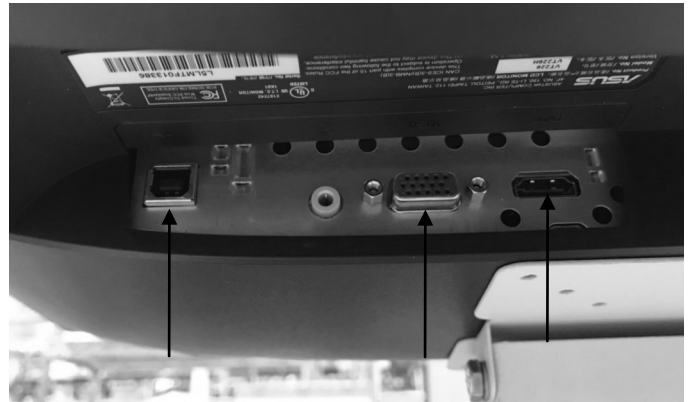


11) Plug the power cord into the port on the back of the top rear cover of the StatSafe.

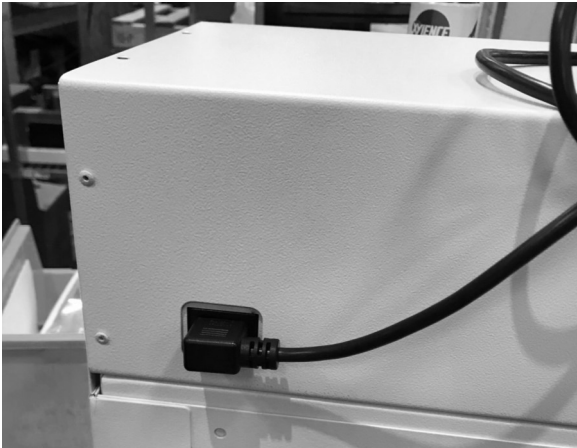

12) See separate Scanner Install Instructions.

13) Locate the two threads on the back of the StatSafe. Screw the antennas onto these. Also install the two rubber bumpers. One on each side of the cart.

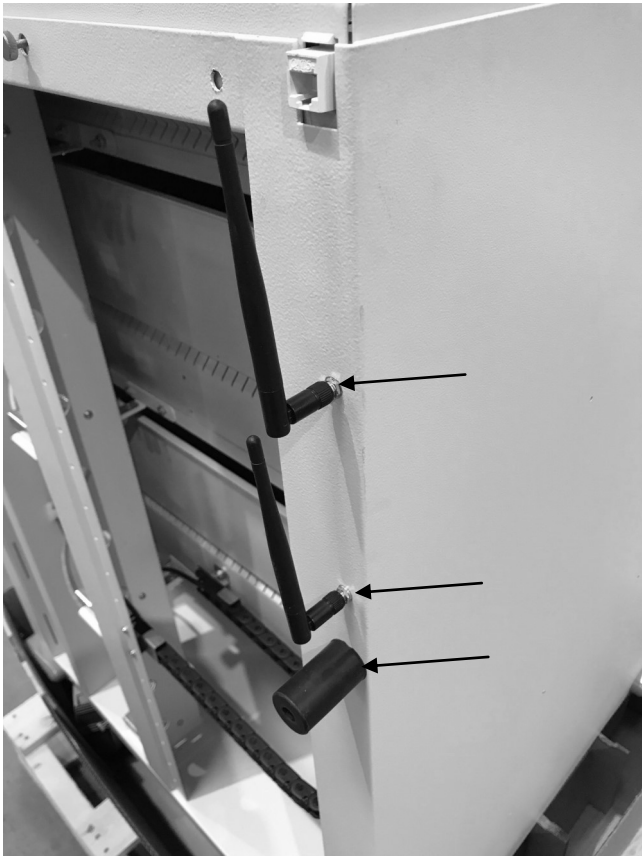

14) Your StatSafe is now completely assembled.

