



AUTOADAPT INVENTORY SYSTEM

**OPEN MATRIX AND PORTAL SETUP
FOR MOST COMMON PACKAGING**

6"x9" PUNCH CARDS

DRAWER: 9"

CAPACITY: 140


When utilizing the packaging above we recommend the drawer and portal setup shown below.

DRAWER SETUP

LABEL DESIGNATION

2A

DRAWER ROW

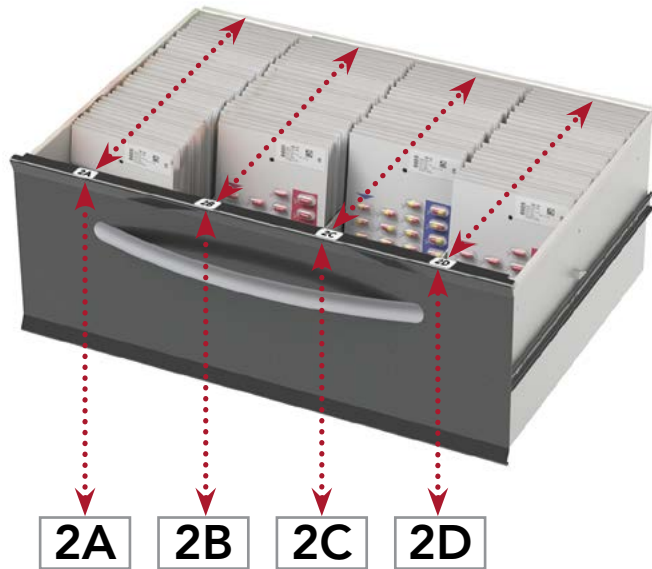

PORTAL SETUP

In the Cart Setup, go to Location Configuration

Under the designated Drawer Number enter values for:

- Rows in the Drawer
- Max Per Row

A screenshot of a web portal interface for 'Location Configuration'. A yellow box highlights the following fields:
Drawer Number: 2
Rows in Drawer: 4
Max Per Row: 35
Other visible fields include: Relay Output (1), Relay Timeout (8.00), Board ID (A), Height (3), and Cube Board ID. There are also checkboxes for 'Bin M...', 'Separ...', 'Requir...', 'Requir...', 'Super...', 'Wide B...', 'Cube B...', and 'Use...'. A 'DELETE LOCATION' button is at the top.

For Further Assistance Please Call or Email Us During Normal Business Hours

855.MED.CART #2 | support@phoenixlhc.com

3"x9" PUNCH CARDS

DRAWER: 9"

CAPACITY: 245


When utilizing the packaging above we recommend the drawer and portal setup shown below.

DRAWER SETUP

LABEL DESIGNATION

2A

DRAWER ROW

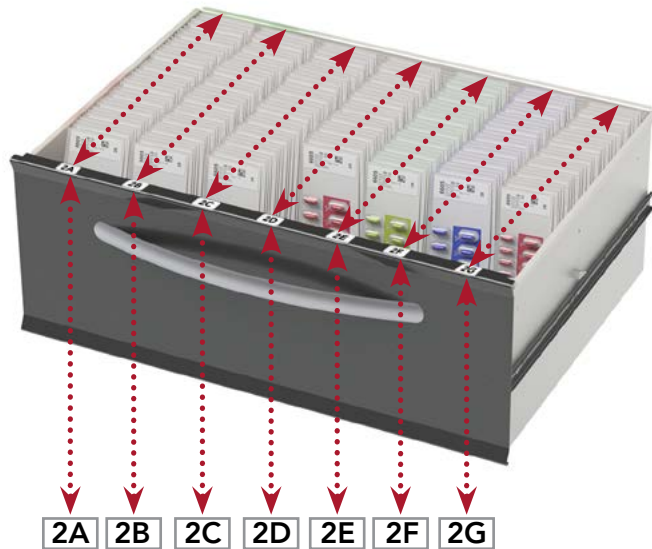

PORTAL SETUP

In the Cart Setup, go to Location Configuration

Under the designated Drawer Number enter values for:

- Rows in the Drawer
- Max Per Row

Drawer Number	2
Rows in Drawer	7
Max Per Row	35

Location Configuration

DELETE LOCATION

Drawer Number	2	<input type="checkbox"/> Bin Mo
Rows in Drawer	7	Bin Name
Max Per Row	35	<input type="checkbox"/> Separ
Relay Output	1	Location De
Relay Timeout	8.00	<input checked="" type="checkbox"/> Requir
Board ID	A	<input type="checkbox"/> Requir
Height	3	<input type="checkbox"/> Requir
Cube Board ID		<input type="checkbox"/> Super
		<input type="checkbox"/> Wide B
		<input type="checkbox"/> Cube B
		<input type="checkbox"/> Use
		Led Outp
		<input checked="" type="checkbox"/> P
		<input type="checkbox"/> B

For Further Assistance Please Call or Email Us During Normal Business Hours

855.MED.CART #2 | support@phoenixlhc.com

BULK STORAGE

DRAWER: 9"
CAPACITY: 28


When utilizing the packaging above we recommend the drawer and portal setup shown below.

DRAWER SETUP

LABEL DESIGNATION

2A

DRAWER ROW

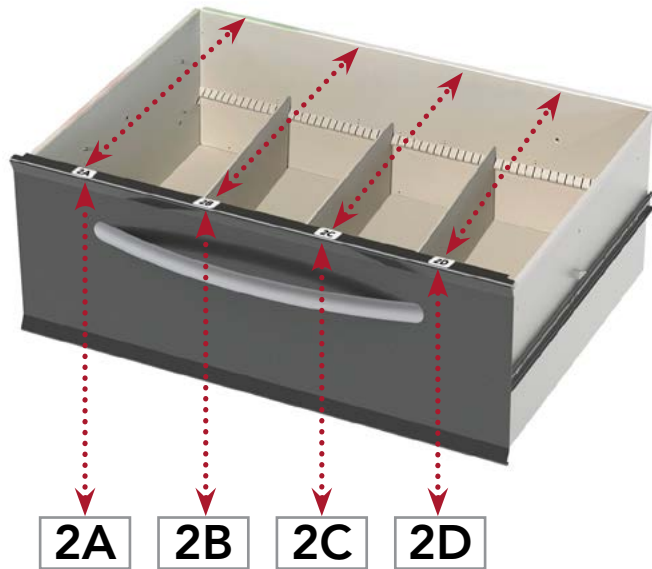

PORTAL SETUP

In the Cart Setup, go to Location Configuration

Under the designated Drawer Number enter values for:

- Rows in the Drawer
- Max Per Row

Drawer Number	2
Rows in Drawer	4
Max Per Row	7

Location Configuration	
DELETE LOCATION	
Drawer Number	2
Rows in Drawer	4
Max Per Row	7
Relay Output	1
Relay Timeout	8.00
Board ID	A
Height	3
Cube Board ID	

For Further Assistance Please Call or Email Us During Normal Business Hours

855.MED.CART #2 | support@phoenixlhc.com

MATCHBOOK CARDS, 16 DOSE (OP or Optipak)

DRAWER: 3"
CAPACITY: 80


When utilizing the packaging above we recommend the drawer and portal setup shown below.

DRAWER SETUP

LABEL DESIGNATION

2A

DRAWER ROW

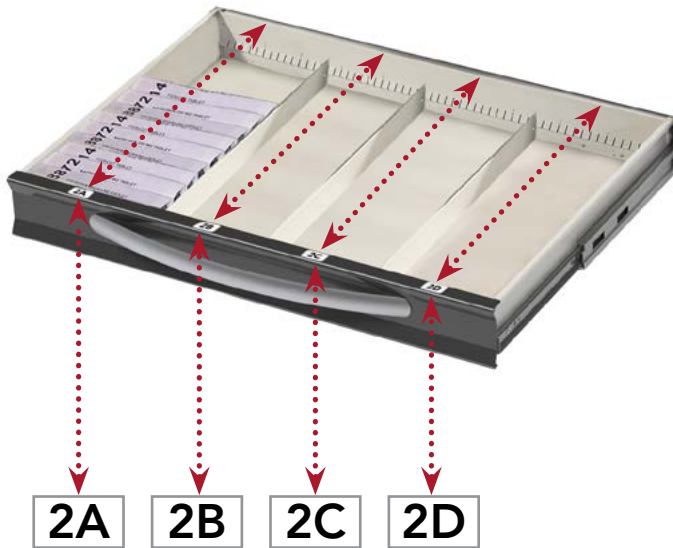

PORTAL SETUP

In the Cart Setup, go to Location Configuration

Under the designated Drawer Number enter values for:

- Rows in the Drawer
- Max Per Row

Location Configuration

DELETE LOCATION

Drawer Number: 2

Rows in Drawer: 4

Max Per Row: 20

Relay Output: 1

Relay Timeout: 8.00

Board ID: A

Height: 3

Cube Board ID:

Bin M...

Bin Name

Separ...

Location De...

Requir...

Requir...

Requir...

Super...

Wide B...

Cube B...

Use...

Led Outp...

P...

For Further Assistance Please Call or Email Us During Normal Business Hours

855.MED.CART #2 | support@phoenixlhc.com

MATCHBOOK CARDS, 8 DOSE (OP or Optipak)

DRAWER: 3"

CAPACITY: 160


When utilizing the packaging above we recommend the drawer and portal setup shown below.

DRAWER SETUP

LABEL DESIGNATION

2A

DRAWER ROW

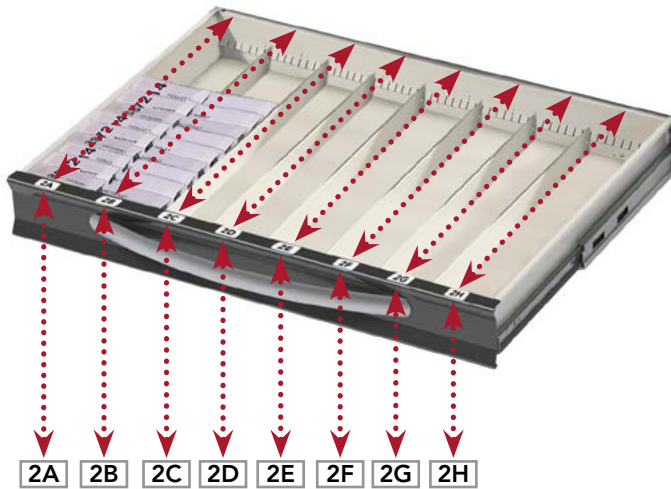

PORTAL SETUP

In the Cart Setup, go to Location Configuration

Under the designated Drawer Number enter values for:

- Rows in the Drawer
- Max Per Row

Location Configuration

DELETE LOCATION

Drawer Number
2

Rows in Drawer
8

Max Per Row
20

Relay Output
1

Relay Timeout
8.00

Board ID
A

Height
3

Cube Board ID

Bin M
Bin Name
 Separ
Location De
 Requir
 Requir
 Requir
 Super
 Wide B
 Cube B
 Use
Led Outp
 P
 B

For Further Assistance Please Call or Email Us During Normal Business Hours

855.MED.CART #2 | support@phoenixlhc.com

PHOENIX LTC MEDIUM BOX

DRAWER: 3"
CAPACITY: 84


When utilizing the packaging above we recommend the drawer and portal setup shown below.

DRAWER SETUP

LABEL DESIGNATION

2A

DRAWER ROW

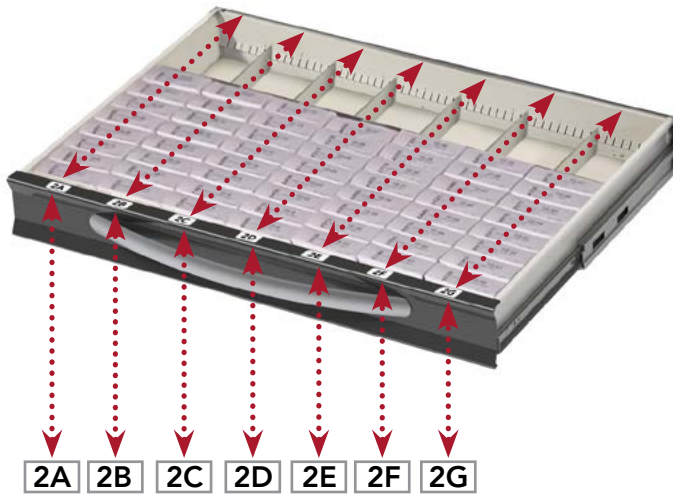

PORTAL SETUP

In the Cart Setup, go to Location Configuration

Under the designated Drawer Number enter values for:

- Rows in the Drawer
- Max Per Row

Drawer Number
2

Rows in Drawer
7

Max Per Row
12

Location Configuration

DELETE LOCATION

Drawer Number
2

Rows in Drawer
7

Max Per Row
12

Relay Output
1

Relay Timeout
8.00

Board ID
A

Height
3

Cube Board ID

Bin Mo
Bin Name
 Separ
Location De
 Requir
 Requir
 Requir
 Super
 Wide B
 Cube B
 Use
Led Outp
 P
 B

For Further Assistance Please Call or Email Us During Normal Business Hours

855.MED.CART #2 | support@phoenixltc.com

BULK STORAGE (EpiPens, Glucagon, etc.)

DRAWER: 3"

CAPACITY: 25


When utilizing the packaging above we recommend the drawer and portal setup shown below.

DRAWER SETUP

LABEL DESIGNATION

2A

DRAWER ROW

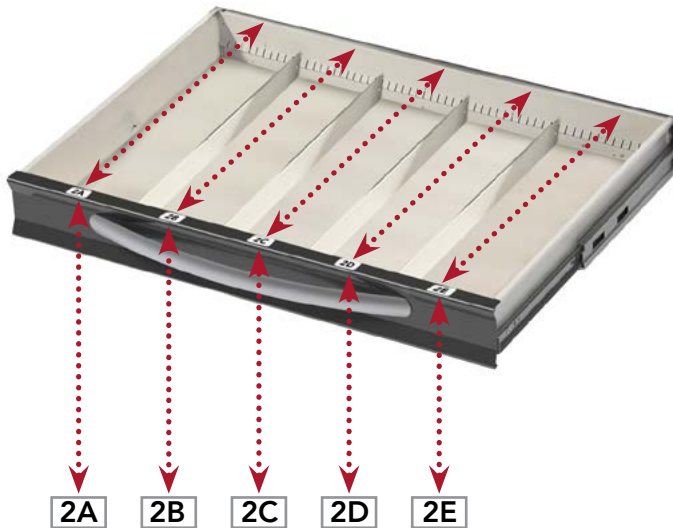

PORTAL SETUP

In the Cart Setup, go to Location Configuration

Under the designated Drawer Number enter values for:

- Rows in the Drawer
- Max Per Row

Drawer Number	2
Rows in Drawer	5
Max Per Row	5

Location Configuration

DELETE LOCATION

Drawer Number	2	<input type="checkbox"/> Bin M...
Rows in Drawer	5	Bin Name
Max Per Row	5	<input type="checkbox"/> Separ...
Relay Output	1	Location De
Relay Timeout	8.00	<input checked="" type="checkbox"/> Requir
Board ID	A	<input type="checkbox"/> Requir
Height	3	<input type="checkbox"/> Requir
Cube Board ID		<input type="checkbox"/> Super
		<input type="checkbox"/> Wide B
		<input type="checkbox"/> Cube B
		<input type="checkbox"/> Use
		Led Outp
		<input checked="" type="checkbox"/> P

For Further Assistance Please Call or Email Us During Normal Business Hours

855.MED.CART #2 | support@phoenixlhc.com