




PHOENIX LTC
MED CARTS



STATSAFE
Automated Medication Management

**STATSAFE PIONEER KIOSK
UPG WIFI KIT**

WWW.PHOENIXLTC.COM

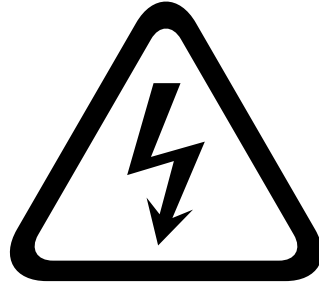

WARNING!
RISK OF ELECTRICAL SHOCK

**PLEASE CONTACT TECHNICAL SUPPORT IF
YOU ARE EXPERIENCING ANY DIFFICULTY
DURING THE TABLET CONVERSION**

855.633.2278 EXT. 2

*FOR COLOR INSTRUCTIONS
SCAN THE QR CODE BELOW*




PART 1: REMOVE YOUR CURRENT TABLET AND DOCKING STATION

1. Using the spanner screwdriver (*provided*), remove the 2 spanner screws.

Remove the front bezel and back weldment.


Remove the old tablet.

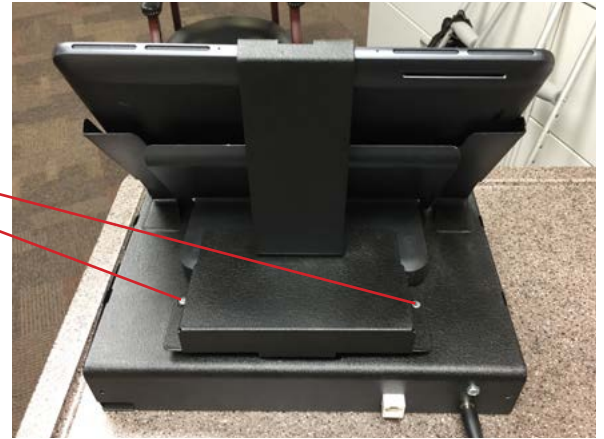

2. Disconnect the 3 cords that are connected to the Docking Station.



3. Follow the power cord, unscrew from the terminal and remove.

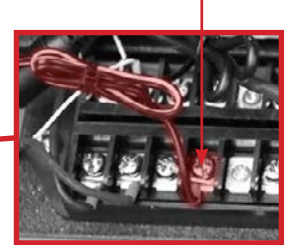

4. Once all cords are disconnected, remove the Docking Station from the top of the kiosk base.

Your kiosk base is now ready to have the new tablet installed.



PARTS AND DESCRIPTIONS

PLEASE NOTE: SOME OF THESE PARTS HAVE BEEN PRE-ASSEMBLED FOR YOUR CONVENIENCE


- 1** CABLE USB 3.0
3 HUB W/GBE
ADAPTER
32-C-USB-300



- 2** CABLE USB MALE
A-FEM A GOLD 10FT
32-C-USB-30



- 3** BACK PANEL
WELDMENT
T19824A

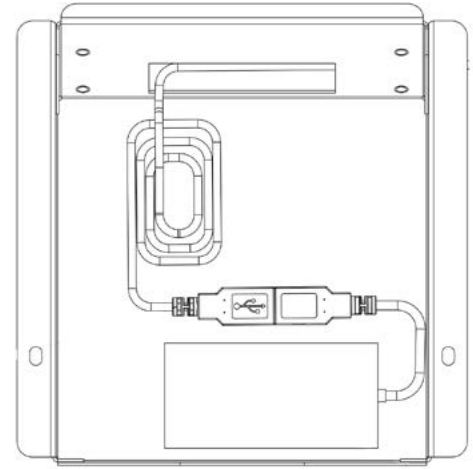

- 4** PIONEER
CHARGER
39-5200-1C



- 5** SPANNER SCREW
8-31 x 3/8
41-SPN-8-25



- 6** SPANNER SCREW
6-32 x 1/4
41-SPN-6-25



- 7** 8-32 KEPS NUT
42-KEPS-8-10



- 8** VELCRO 2"
5480-6




- 9** PIONEER DASH/T3
2GB 10.1" TAB
39-5200-1-PD



- 10** BEZEL, TABLET SHELL,
PIONEER
A17331A1



- 11** BACK COVER
A17330

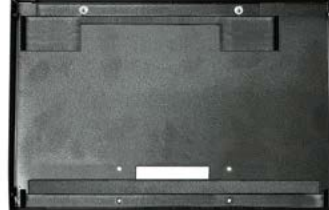

- 12** POWER CORD
32-PC-0030



- 13** USB 4"
32-C-USB-35



Your new tablet
needs to be charged
before installing.
Please remove the
power supply and
tablet from the box
and allow to charge
for 15-30 minutes
prior to installing.

TOOLS INCLUDED



#6 Spanner Bit
99-3972



#8 Spanner Bit
99-3972-1



Nut Driver 1/4 Shank
Power Bit Set
99-3467



Screwdriver 6 in 1
99-5729-1

PART 2: SETTING UP YOUR TABLET

CONNECT THE USB HUB

1. Locate the Cable USB 3.0 3 Hub w/GBE Adapter underneath the back cover and secure with 2"x 2.5" velcro square (5480-6).



2. Insert the 2 USB cords that were plugged into the docking station into the Cable USB 3.0 3 Hub w/GBE Adapter.




3. Insert the Ethernet Cable into the hub.



ROUTING THE POWER


1. Route the supplied charger through the bottom base and into the back cover.

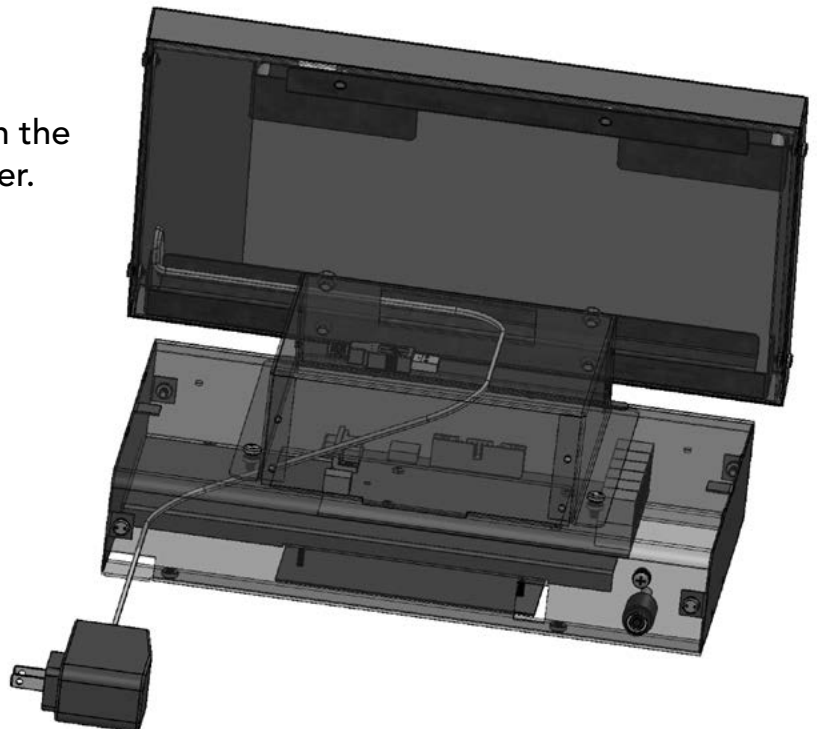

INSTALLING THE PIONEER TABLET

1. Take the back cover and place it onto the kiosk base, making sure all of the exposed wires are fully covered.


Locate the 2 pre-drilled holes on the back cover and line them up with the pre-drilled holes on the kiosk base. Secure into place with spanner screws using the spanner screwdriver.

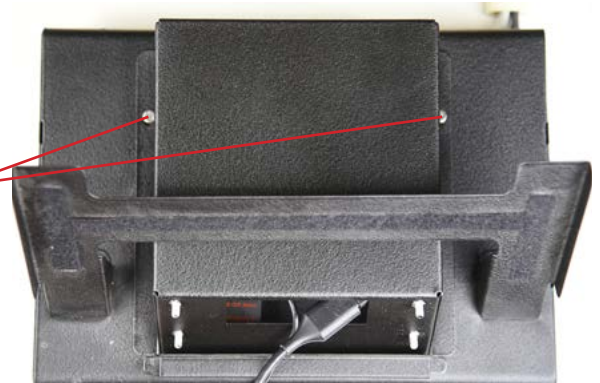

2. Pull the Harness Terminal Male Cable and the USB Cable through the opening in the back tablet weldment.

Mount the Back Tablet Weldment into place by lining up the 4 pre-drilled holes to the spanner screws on the back cover.

Secure with KEPS Nuts.

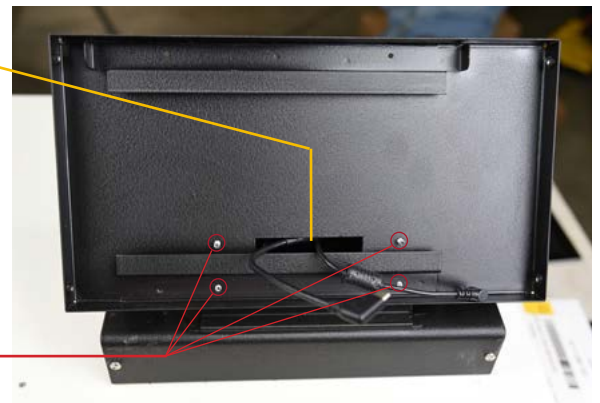

3. Take the two cables and pull them to the right hand side.



4. Place the new tablet into the slots that are welded inside the Back Tablet Weldment (*the power button should be on the top*)



5. Plug the two cables into their ports on the tablet.



6. Take the Front Bezel and place onto the Back Tablet Weldment to enclose the tablet.



7. Secure the Front Bezel into place with spanner screws.



Please call Tech Support immediately to have your Cart ID assigned to the new tablet.

855.633.2278

Once you have the Cart ID re-assigned your conversion is complete and your tablet is ready for use!



PHOENIX LTC
MED CARTS

WWW.PHOENIXLTC.COM