




PHOENIX LTC
MED CARTS



STATSAFE
Automated Medication Management

**CONVERTING STATSAFE FROM
LENOVO TO PIONEER TABLET**

WWW.PHOENIXLTC.COM

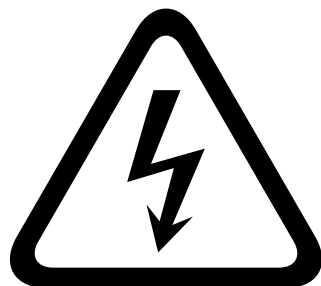

WARNING!
RISK OF ELECTRICAL SHOCK

**PLEASE CONTACT TECHNICAL SUPPORT IF
YOU ARE EXPERIENCING ANY DIFFICULTY
DURING THE TABLET CONVERSION**

855.633.2278 EXT. 2

*FOR COLOR INSTRUCTIONS
SCAN THE QR CODE BELOW*



PARTS AND DESCRIPTIONS

PLEASE NOTE: SOME OF THESE PARTS HAVE BEEN PRE-ASSEMBLED FOR YOUR CONVENIENCE



1 USB 4"
32-C-USB-35



2 CABLE USB
MALE A-FEM A
GOLD 10FT
32-C-USB-30



3 CABLE USB
3.0 3 HUB W/
GBE ADAPTER
32-C-USB-300




4 PIONEER
CHARGER
39-5200-1C



5 SPANNER SCREW
6-32 x 1/4
41-SPN-6-25



6 PIONEER DASH/T3
2GB 10.1" TAB
39-5200-1-PD

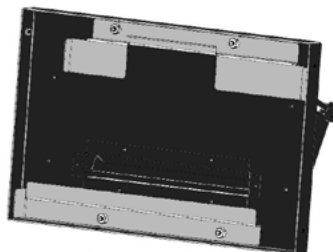

7 KIOSK, PIONEERII TILT
ASSEMBLY
A17229A1



8 BEZEL, TABLET
SHELL
I-1114A1

Your new tablet needs to be charged before installing. Please remove the power supply and tablet from the box and allow to charge for 15-30 minutes prior to installing.

TOOLS INCLUDED



#6 Spanner Bit
99-3972



#8 Spanner Bit
99-3972-1




Nut Driver 1/4 Shank
Power Bit Set
99-3467



Screwdriver 6 in 1
99-5729-1

PART 1: CHARGE THE PIONEER TABLET BEFORE INSTALLING

1. Remove the 4 spanner screws on the sides of the tablet assembly using the #6 Spanner Bit.

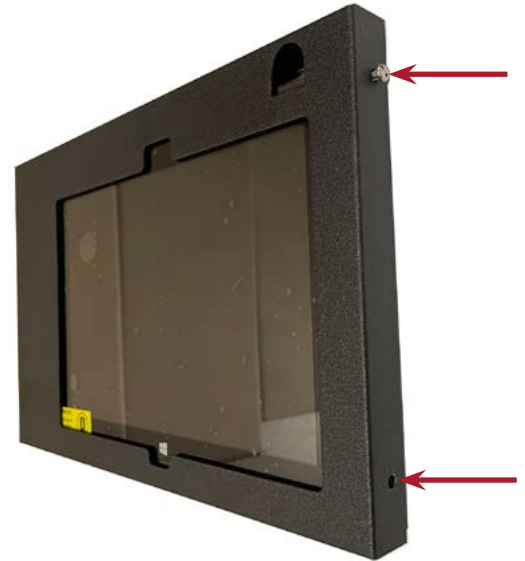

2. Remove the tablet from the assembly and charge.




PART 2: REMOVE OLD TABLET AND TILT ASSEMBLY

1. Use your key to unlock and remove the rear cover panel.



2. Use the security spanner bit and driver to remove the (4x) #6 security screws that hold the front bezel in place.

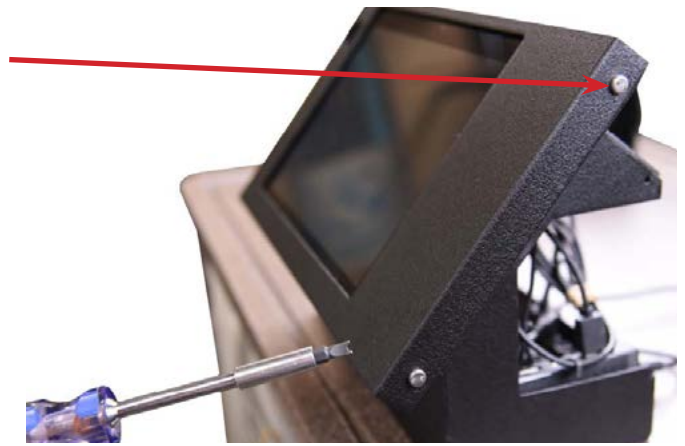

3. Remove the front bezel.




4. Disconnect USB adapter and power supply and remove the Lenovo tablet from the kiosk.




5. Remove the USB adapter and the Startech hub that is inside the base of the kiosk.



6. Move all of the USB cables plugged into the Startech to the new Teknet HUB.

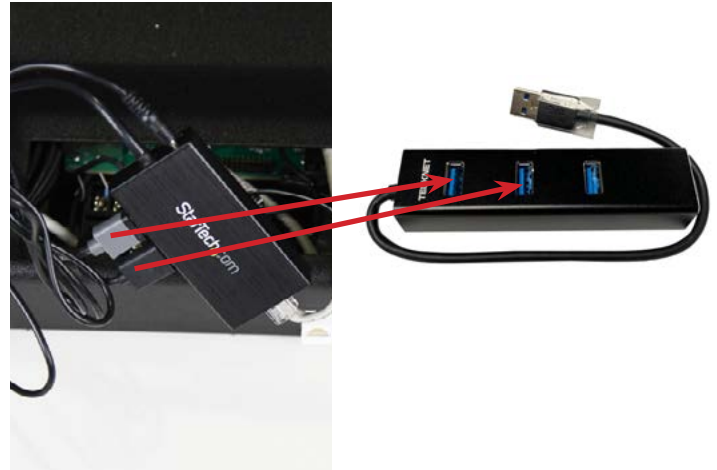

7. Using a socket wrench, remove the (4x) #8 nylock nuts that hold the tilt assembly in place.

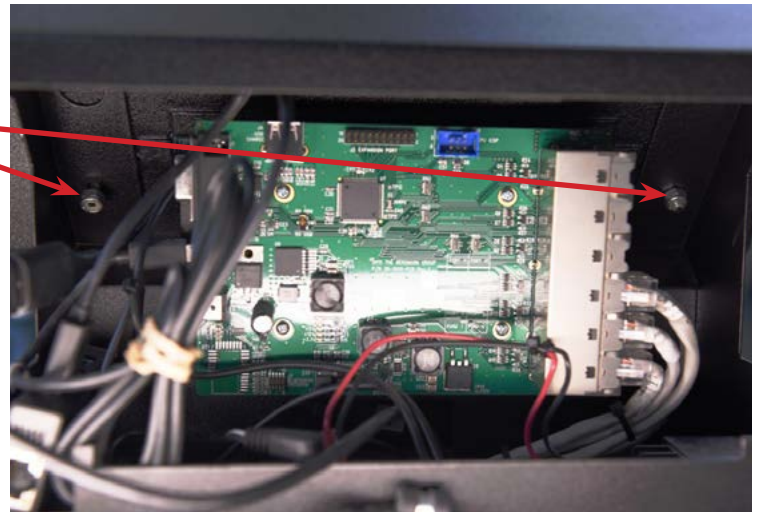

8. Remove the tilt assembly. Keep the nylock nuts for later use.

PART 3: INSTALL THE NEW TILT ASSEMBLY AND PIONEER TABLET


1. Assemble the following parts:



USB 4"
Part 1
32-C-USB-35




CABLE USB MALE
A-FEM A GOLD 10FT
Part 2
32-C-USB-30

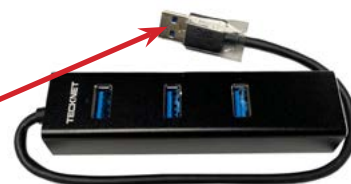


CABLE USB 3.0 3 HUB W/
GBE ADAPTER
Part 3
32-C-USB-300

2. Feed the USB assembly (Part 1 head first) and Pioneer charger (Part 4) through the back of the kiosk base and through the back of the kiosk tilt assembly.

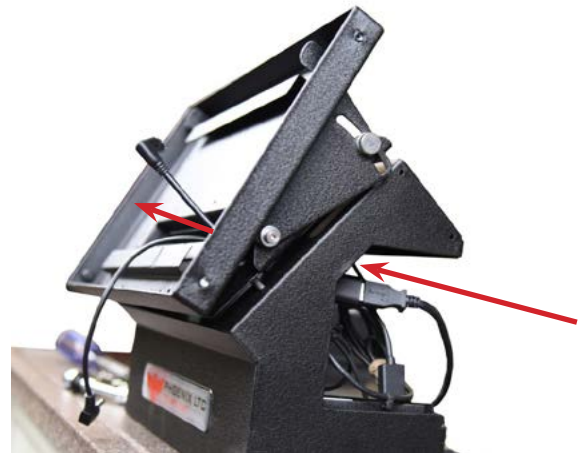


3. Attach the kiosk tilt assembly (Part 7) to the kiosk base by inserting the 4 threaded posts through the holes on the kiosk base.

Secure by attaching the nylock nuts that were removed earlier.

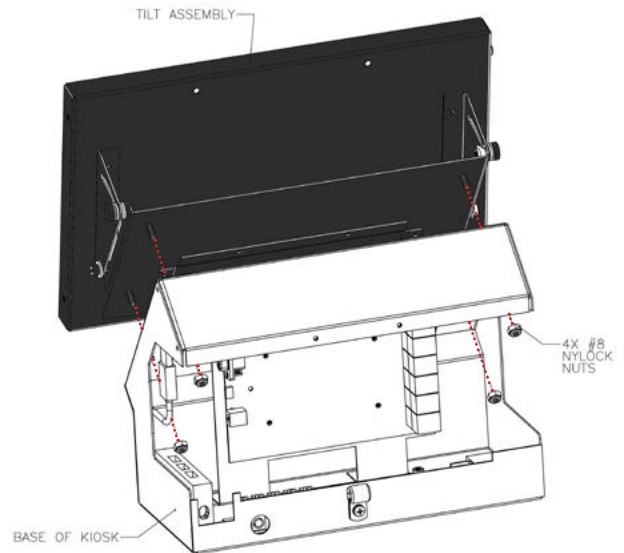

4. Plug the USB 4" (Part 1) and Pioneer charger into the tablet.

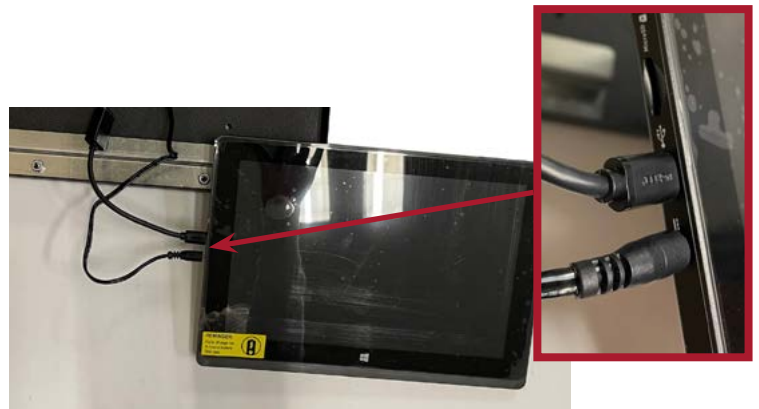

5. Place the tablet into the kiosk tilt assembly tucking the two cords behind it.



6. Place the tablet shell onto the kiosk tilt assembly. Secure with 4 spanner screws.



7. Place the back cover on the kiosk plate, carefully placing the Pioneer charger cord between them.

Gently close and lock the cover.

8. Plug the Pioneer charger into the wall in addition to the original power supply.



Please call Tech Support immediately to have your Cart ID assigned to the new tablet.

855.633.2278

Once you have the Cart ID re-assigned your conversion is complete and your tablet is ready for use!



PHOENIX LTC
MED CARTS

WWW.PHOENIXLTC.COM