


# STATSAFE

Automated Medication Management

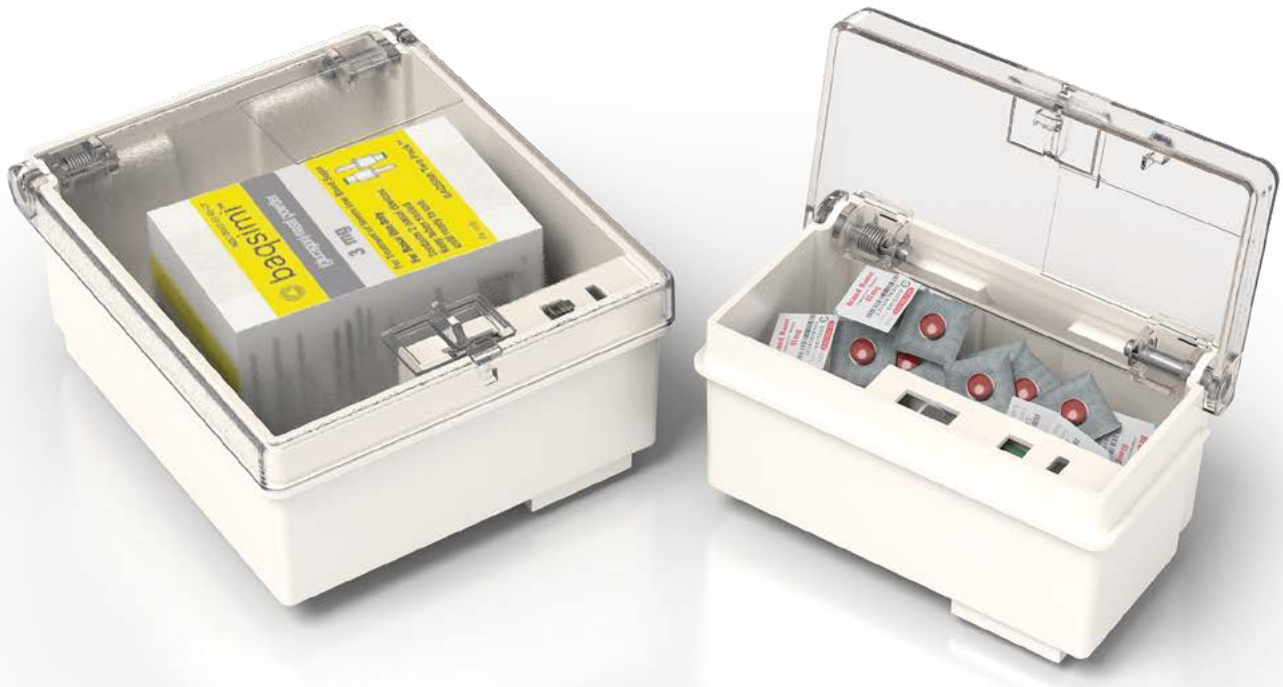


## HOW TO RETURN CUBES WITH INVENTORY IN THEM

When a cube is returned to the pharmacy with inventory still loaded in it follow these steps to open the cube and remove any remaining inventory.

1. Log into the Statcube loader program and select "Check In Returned Cubes."



2. A list of available Statcubes will appear to return and check back in that have remaining inventory in them:

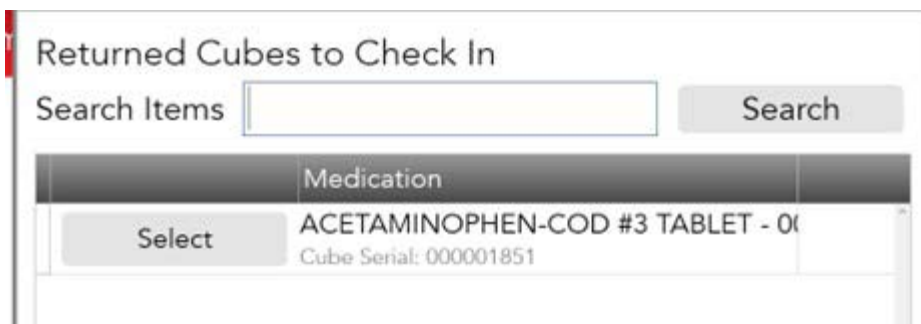

3. Follow the screen prompts, scan the Statcube and load it onto the programming station.



4. The lid will open allowing you access to confirm the Qty returned and remove the medication.



The cube will release with the lid open and is ready to be used again with new inventory