



PLXMNT-3 (Arm Mount)

Entire Kit Shown Here



A17012A4: STAND BRACKET
with 56-1116; Plug, Black 1-3/8"
OD 1.21-1.30ID




41- PHMS- 25-35; PAN HD PHIL
MS 1/4-20 x 1.500



M14095; PLUG W/SQR CUTOUT
ERGOMNT WOOD
(thicker for wood top)




M14095A7; PLUG W/SQR CUTOUT
ERGONMNT PLASTIC
(thinner plastic for top)

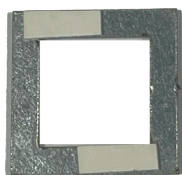


T19866; PLX MOUNT DRILL JIG
with 63-1320, TAPE DUBL KOTE
WHT 1/32 x 5/8 T.L



P-ARM1 Supplied Parts list

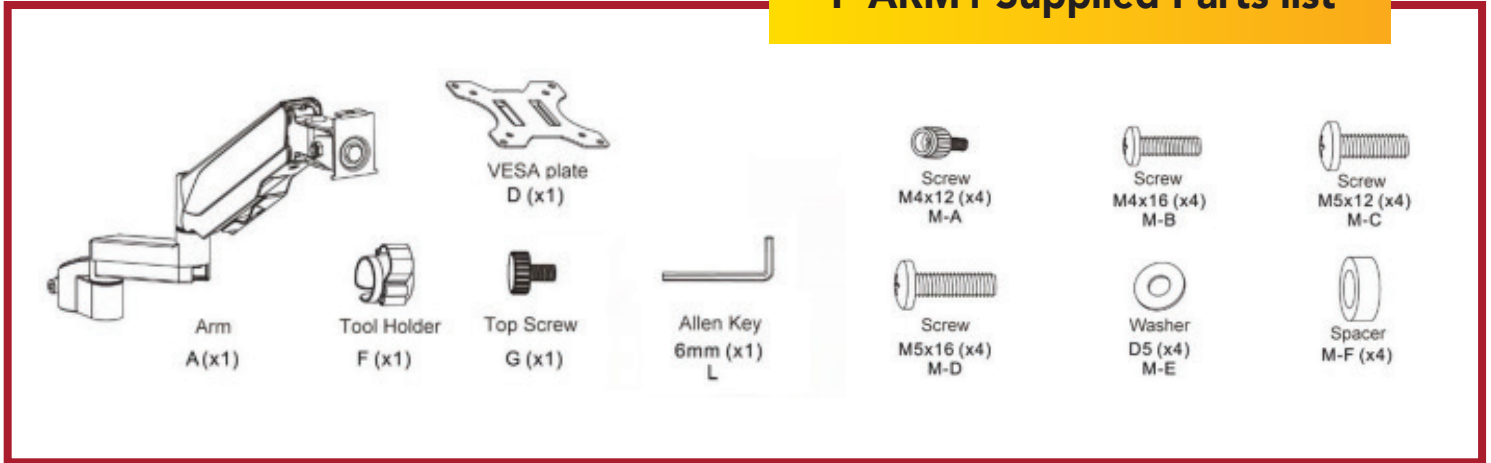


Supplied Parts List that will be used.

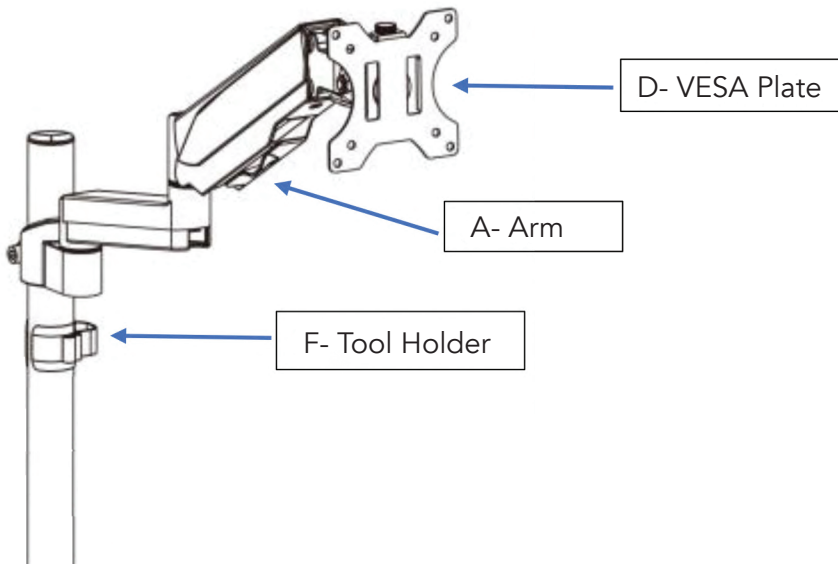


- A- Arm
- D- VESA Plate
- F- Tool Holder
- G- Top Screw
- L- Allen Key 6mm
- M-A- Screw M4x12
- M-B- Screw M4x16
- M-C- Screw M5x12
- M-D- Screw M5x16
- M-E- Washer D5
- M-F- Spacer

To attach arm, start from step 3 of install directions.

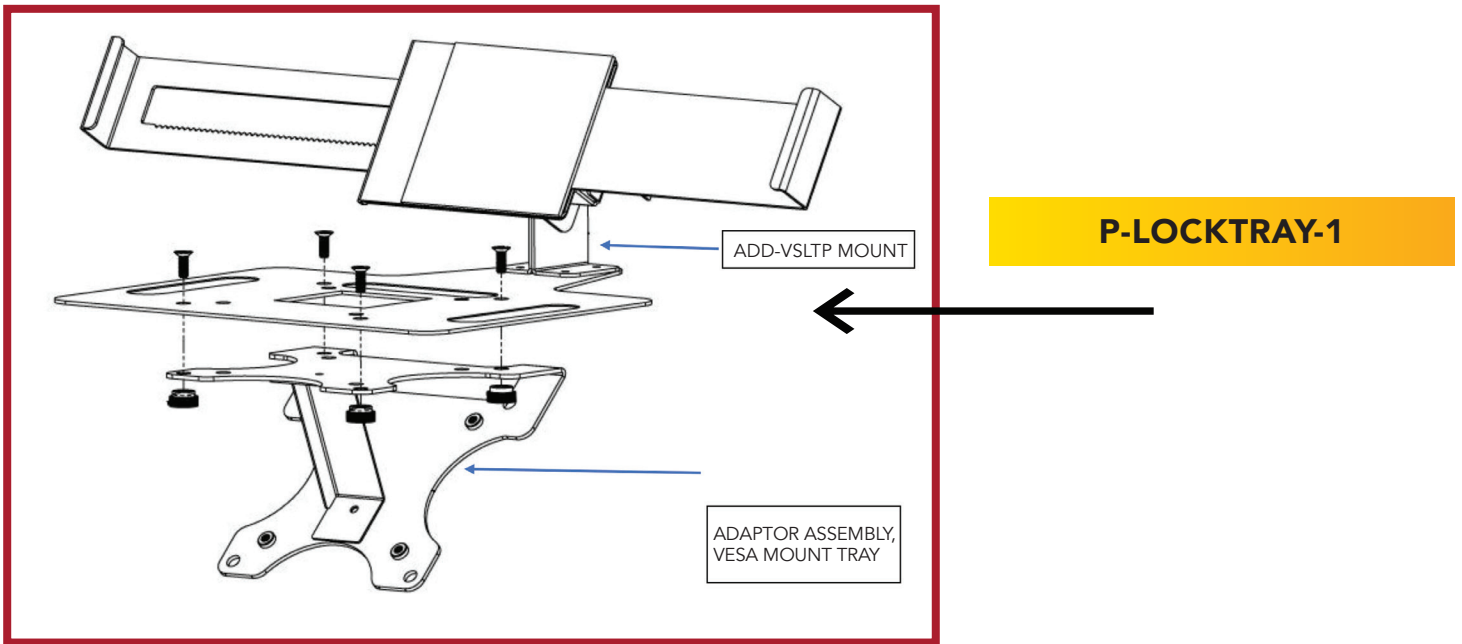


- 1 You will attach the ADD-VSLTP MOUNT to the ADAPTOR ASSEMBLY VESA MOUNT TRAY. Then from the P-ARM1 kit you will attach the VESA plate (item# D) to the ADAPTOR ASSEMBLY VESA MOUNT TRAY from the PLOCKTRAY-1.

VESA plate

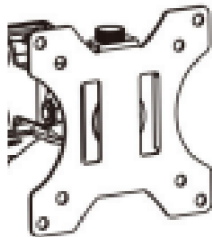


(Comes with P-ARM1)

ADAPTOR ASSEMBLY
VESA MOUNT

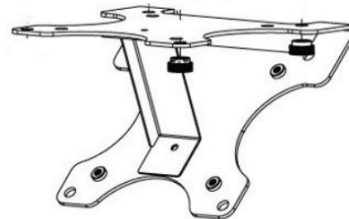

(Comes with P-LOCKTRAY-1)

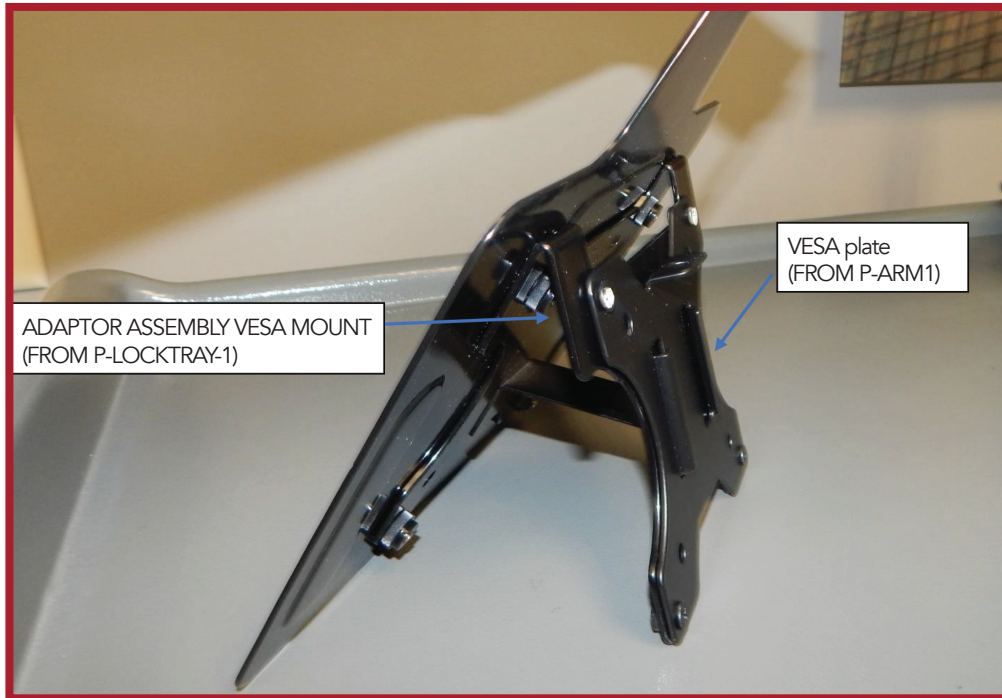

- 2 Once the ADAPTOR ASSEMBLY VESA MOUNT and VESA plate are attached you can then slide the locktray onto the arm.





PHOENIX
MEDICATION MANAGEMENT SOLUTIONS

How to Install the EMAR LOCKTRAY

