



PHOENIX
MEDICATION MANAGEMENT SOLUTIONS

How to Install Cart Handle



P-HRHANDLE (Alum Cart Handle)

Entire Kit Shown Here




A17338 Cart Handle Assembly



4X 41-PHMS-250-20

PAN HEAD PHIL MS 1/4-20 X 3/4"

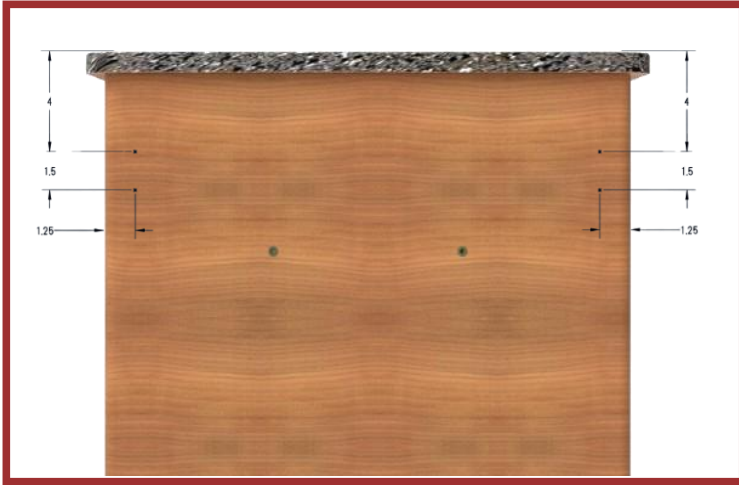

1 Locate the four mounting holes on the left side of your cart.

The approximate locations are:

- 4.0" down from the top
- 1.25" in from each side
- 1.50" spacing between holes

2 To pinpoint the exact location of each hole, press your fingers against the vinyl covering. The vinyl will give slightly where the hole is located compared to the surrounding metal surface





- 3 Once you have confirmed the hole locations, use a small Phillips screwdriver to carefully poke through the vinyl, exposing the mounting holes

Tip: Take care to make only a small opening in the vinyl, just enough to expose the screw holes for a clean finished appearance.

- 4 Install the handle screws into the exposed holes and tighten securely

