


STATSAFE

Automated Medication Dispensing System

By  PHOENIX

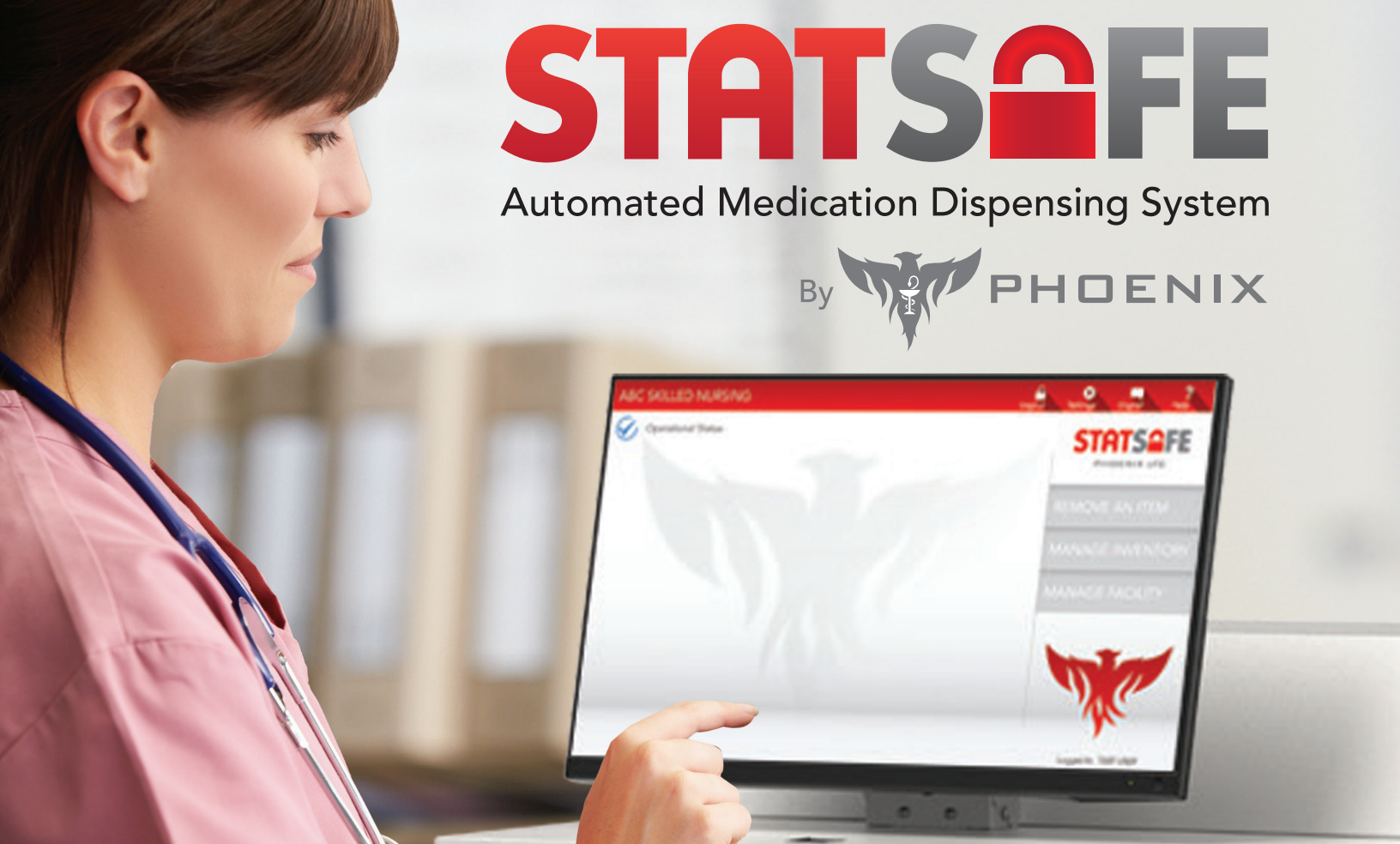

New StatSafe Features – April 2026



PhoenixLTC.com | 855.MED.CART (633.2278)

Bulk Accept – Witness and New Remove Item Settings

Phoenix LTC is excited to offer several new features based on customer feedback and needs. All the below are optional add-on features that can be customized by individual StatSafe.

Each StatSafe cabinet will need to be updated to see these new features. Please reach out to our support team at 1-855-633-2278 ext. 2 and we will be glad to walk you through the process.

New -Bulk Accept of Medications in StatSafe


The new bulk accept of medications feature allows the user to load all items by drawer in one transaction without having to open and close the drawer for each item. This saves time as well as offers a new process for loading Statcube drawers

To add this feature:

- Select the StatSafe you would like to add it to
- Select **Edit Settings** and then the **S2 Settings** tab.
- Select **Allow Bulk Acceptance**

User Directions for Bulk Accept


1. When the user logs in and selects Manage Inventory and Accept Stock they will then be asked to choose Bulk Accept or Single Items

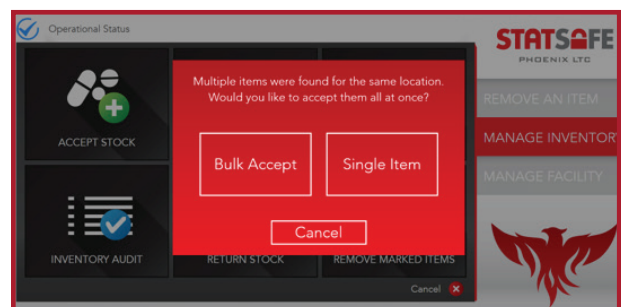

New StatSafe Features

April 2026


- When they select Bulk Accept they will see a list of the drawers with medications to be accepted and the quantity by the drawer.

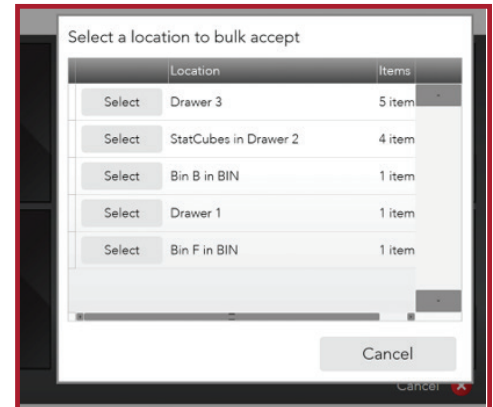

- When accepting items in open matrix drawers, the user will be prompted to enter the PIN code manually or scan the item and verify the quantity, then they can load it into the drawer position.




- This process is repeated with the drawer open until all items for that location have been verified.

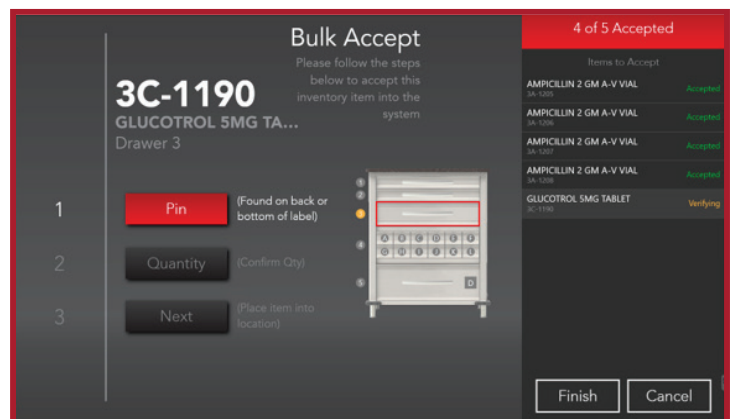

New StatSafe Features



April 2026

- Once all the items have been accepted, the user can close the drawer.

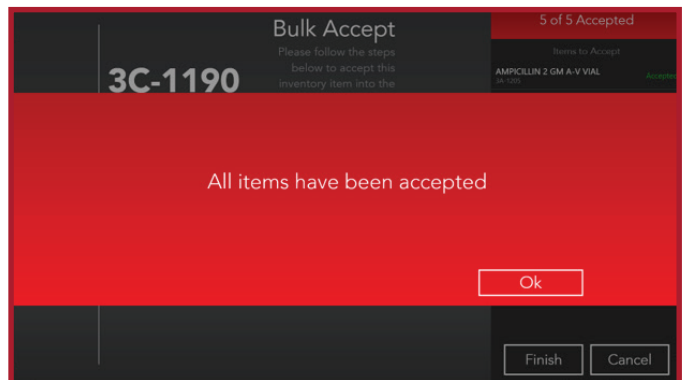

Bulk Accepting Statcubes

- The pharmacy will create and assign the Statcubes for the StatSafe. During that process, locations are chosen, however during the accept process, these locations may change.
- When the user is ready to accept the Statcubes, they will select the Statcube drawer

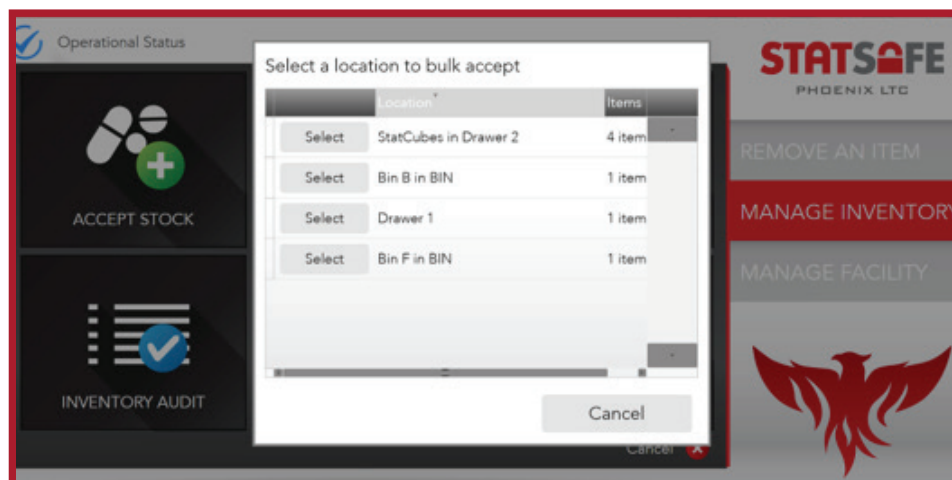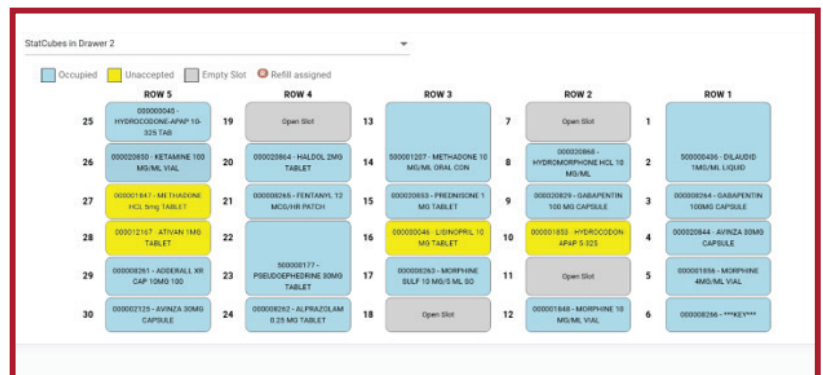


New StatSafe Features


April 2026

- A list of the Statcubes to be accepted will appear on the screen, and all open slots will appear on the screen.

*****Note – If you have assigned the Statcube on top of an existing Statcube, that Statcube will eject from the drawer during this process. Please make sure to remove them.**

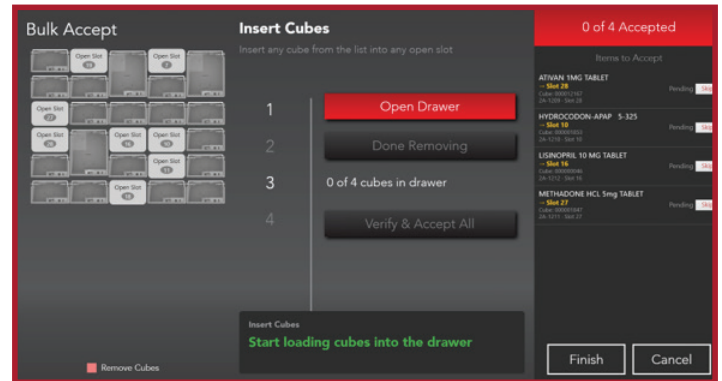

- All you need to do is just place the Statcubes into any open slot, you do not need to scan them, just secure them into open locations.

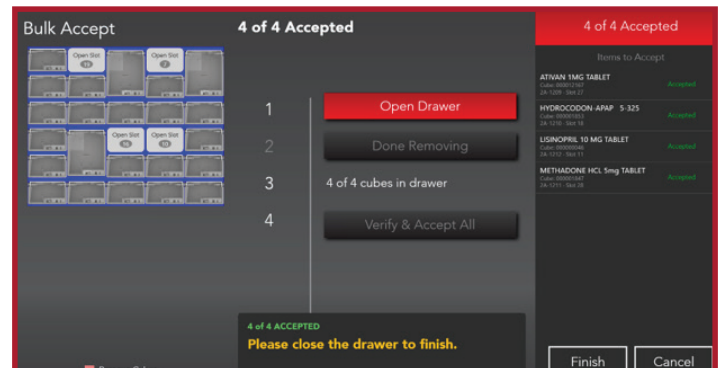

- After the user logs out, the new location will show on the Statcube drawer map in the portal where they were placed.

StatCubes in Drawer 2

Occupied

Unoccupied

Empty Slot

Refill assigned

ROW 5		ROW 4		ROW 3		ROW 2		ROW 1	
25	00000045- HYDROCODONE-APAP 10-325 TAB	19	Open Slot	13		7	Open Slot	1	
26	00000080- NITAMINE 100 MG/ML VIAL	20	00000084- HALDOL 2MG TABLET	14	00001207- METHADONE 10 MG/ML ORAL CON	8	00000098- HYDROCODONE HCL 10 MG/ML	2	00000434- ELAIDID 1MG/ML LIQUID
27	00012187- ATIVAN 1MG TABLET	21	00000005- FENTANYL 12 MG/HR PATCH	15	00000080- PREDNISONE 1 MG TABLET	9	00000029- GABAPENTIN 100 MG CAPSULE	3	00000024- GABAPENTIN 100MG CAPSULE
28	00001047- METHADONE HCL 5mg TABLET	22		16	Open Slot	10	Open Slot	4	00000084- APALZA 30MG CAPSULE
29	00000026- ADDERALL XR CAP 30MG 100	23	00000077- PSEUDOPHEDRINE 30MG TABLET	17	00000028- MORPHINE 30MG 10 MG/5 ML 10	11	00000046- LISINAPRIL 10 MG TABLET	5	00001080- MORPHINE 30MG/ML VIAL
30	00000125- APINZA 30MG CAPSULE	24	00000020- ALPHAZOLAM 0.25 MG TABLET	18	00000180- HYDROCODONE- APAP 5-325	12	00001040- MORPHINE 10 MG/ML VIAL	6	00000026- ***KEY***


New Open Remove Item After Login Setting

To add this feature:

1. Select the StatSafe you would like to add it to
2. Select **Edit Settings** and then the **S2 Settings tab**.
3. Select **Open Remove Item** after Login

- ☐ Force Witness on Waste Function
- ☐ Allow Standard User to Remove Marked Items
- ☐ Allow Bulk Acceptance
- ☒ Open Remove Item after Login
- ☐ Move Witness to After Security Code

4. When this setting is selected, the Remove an Item screen appears at login as soon as the user is logged in.
5. If they are not removing a medication, they can select cancel to go to other functions.

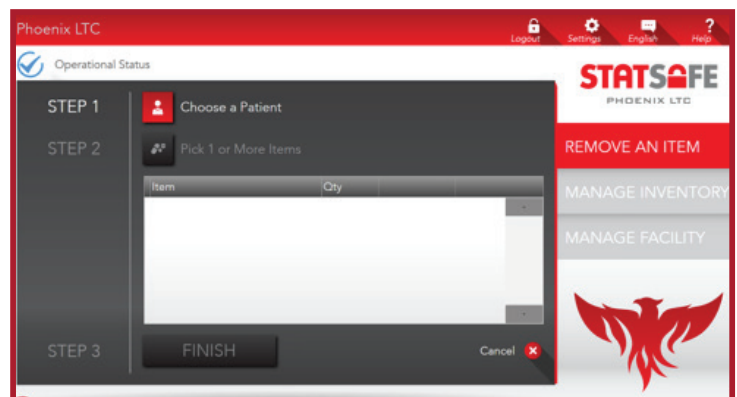

New – Move Witness to After Security Code


To add this feature:

1. Select the StatSafe you would like to add it to
2. Select **Edit Settings** and then the **S2 Settings tab**.
3. Select **Move Witness to After Security Code**


- ☐ Items
- ☐ Allow Bulk Acceptance
- ☐ Open Remove Item after Login
- ☒ Move Witness to After Security Code

New StatSafe Features April 2026


4. Using this setting allows one user to be present to go in and request a medication.

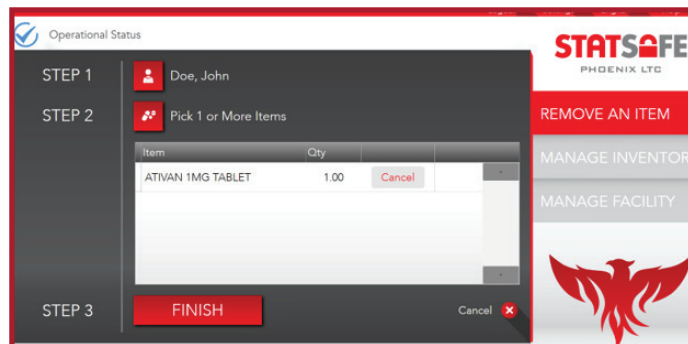

5. It moves the need for a witness and 2nd user until after a security code has been entered.

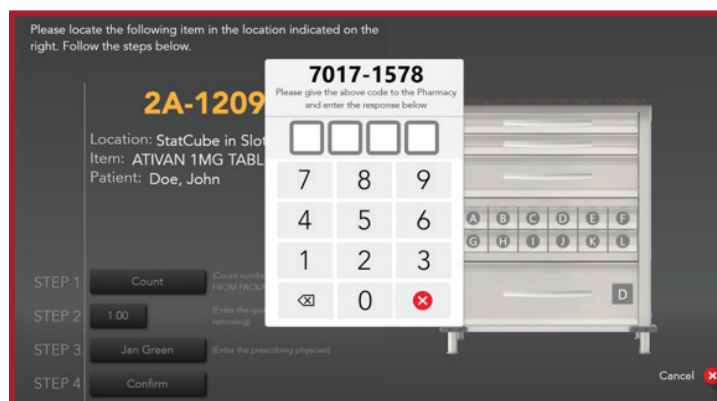

6. After the code has been entered, the witness will need to be present to enter their credentials or biometrics.

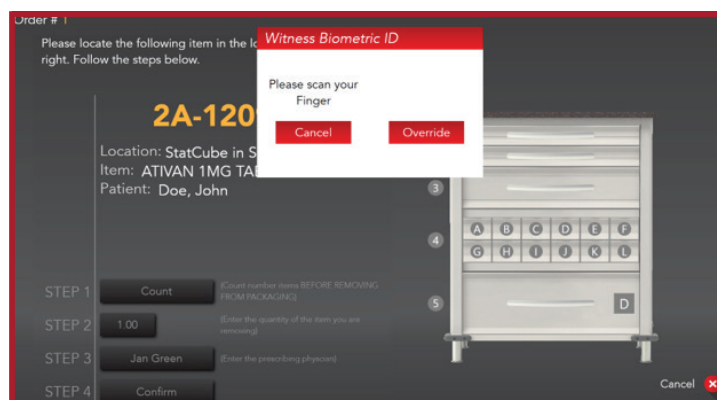

New StatSafe Features April 2026

- The drawer will not open until a Witness user has presented their credentials.

Order # 1

Please locate the following item in the location indicated on the right. Follow the steps below.

2A-1209

Location: StatCube in Slot 27 in Drawer 2
Item: ATIVAN 1MG TABLET
Patient: Doe, John

STEP 1 **Count** (Count number items BEFORE REMOVING FROM PACKAGING)

STEP 2 (Enter the quantity of the item you are removing)

STEP 3 (Enter the prescribing physician)

STEP 4



Cancel 

If you would like more information or training for this new feature,
please contact us at 1-855-633-2278 ext 2 or Support@phoenixltc.com.