


STATSAFE

Automated Medication Dispensing System

By  PHOENIX

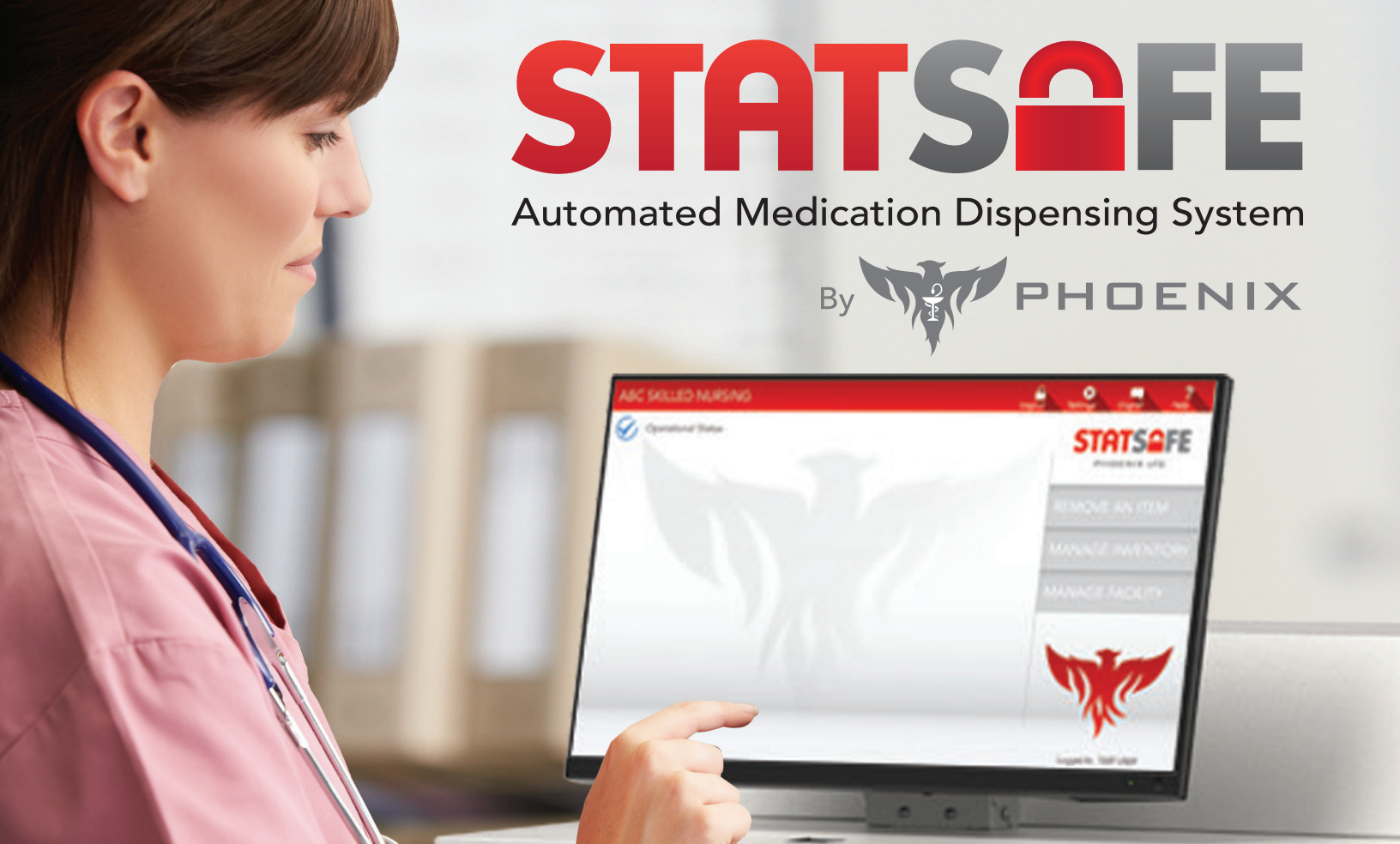

New StatSafe Statcube Lists




PhoenixLTC.com | 855.MED.CART (633.2278)

New StatSafe Statcube Lists


StatSafe now offers a feature that allows you to create a list of items to be filled into Statcubes using a list created by the Pharmacy. This new feature makes it faster to create items for multiple StatSafe's at one time.

- Create a list in the StatSafe Portal of your standard items used with Statcubes
- Use the same list for multiple facilities or adjust for specific medication lists.
- Quickly fill and create Statcubes using list for multiple StatSafes


This new feature can be found in the StatSafe portal under Inventory Libraries and then Manage Cube Lists.

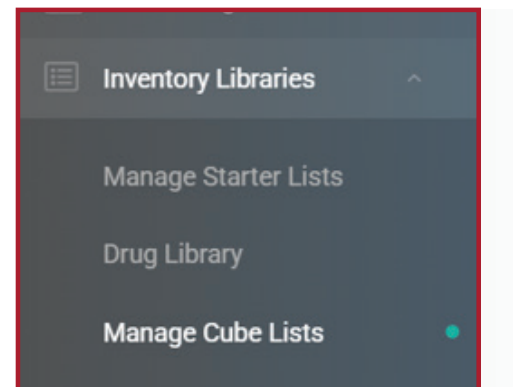


1. Select +Add Cube List to create a new cube list

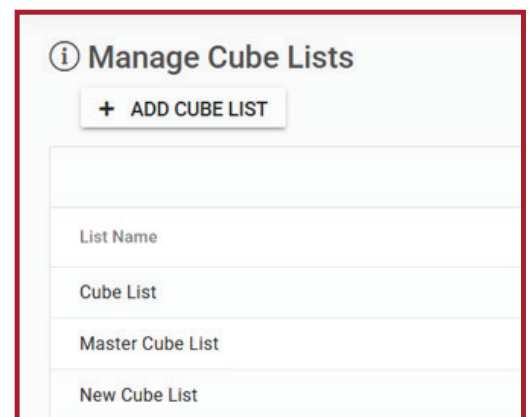A screenshot of the 'Manage Cube Lists' form. At the top, there is an information icon and the title 'Manage Cube Lists'. Below the title is a button labeled '+ ADD CUBE LIST'. Underneath the button are four input fields: 'List Name', 'Cube List', 'Master Cube List', and 'New Cube List'.


2. Create a name for your new list. You can add then each item, or you can paste a list from Excel.

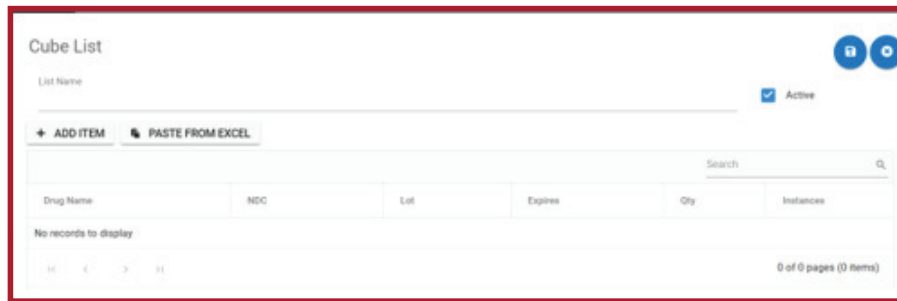

3. To paste your list from Excel, select the Paste from Excel option. Then Download template, fill in the information into the fields, and paste Excel data into space provided.

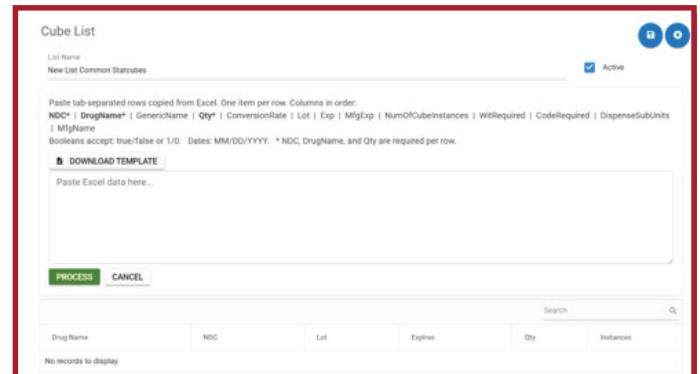


4. The cube list will display the added items and can still be modified or you can more items.

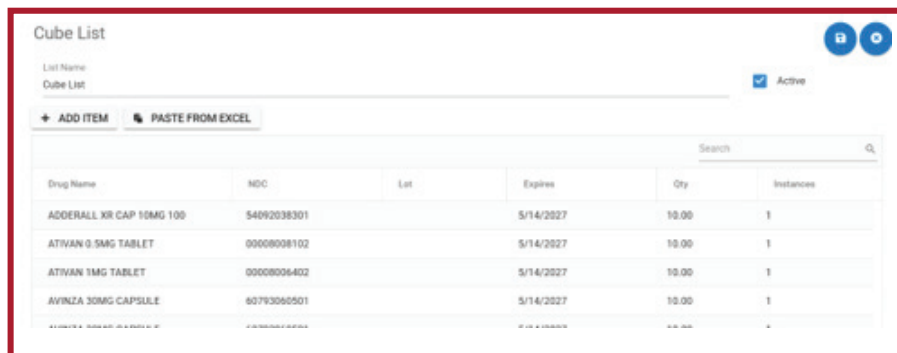


Drug Name	NDC	Lot	Expires	Qty	Instances
ADDERALL XR CAP 10MG 100	54092038301		5/14/2027	10.00	1
ATIVAN 0.5MG TABLET	00008008102		5/14/2027	10.00	1
ATIVAN 1MG TABLET	00008006402		5/14/2027	10.00	1
AVINZA 30MG CAPSULE	60793060501		5/14/2027	10.00	1

New StatSafe Statcube Lists


5. Your new list will not be available in the Statcube Loader program under Create Cubes from Quick List.

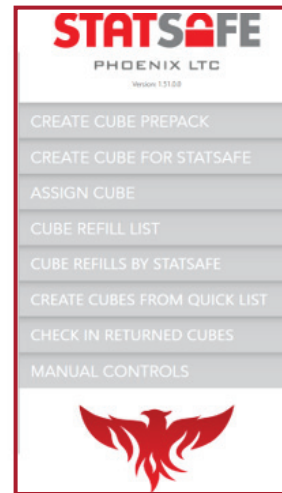

6. Choose Create Selected, then complete the required fields on the Inventory Item screen..

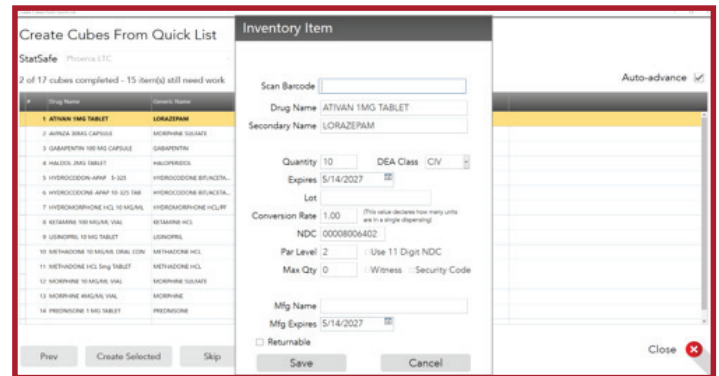

7. You will be prompted to scan and complete the StatCube.

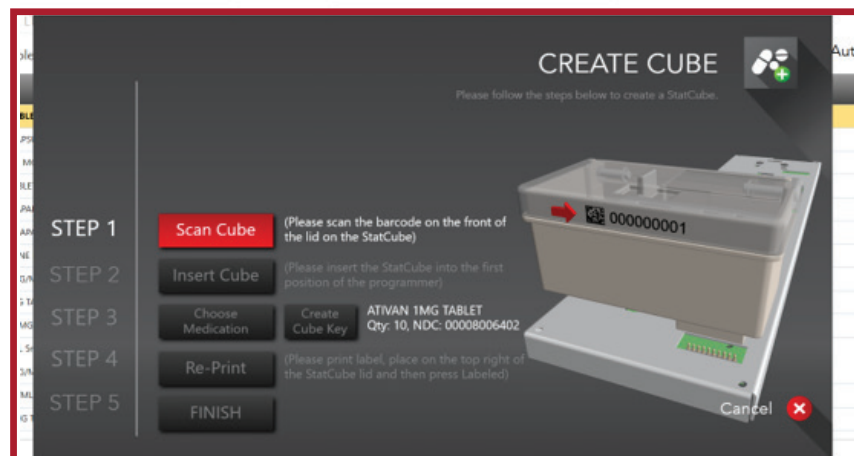

New StatSafe Statcube Lists

8. As you finish each StatCube the system will take you to the next item on the list. You can skip items you do not want to create.

Create Cubes From Quick List

StatSafe Phoenix LTC

3 of 17 cubes completed - 14 item(s) still need work

#	Drug Name	Generic Name
1	AVINZA 30MG CAPSULE	MORPHINE SULFATE
2	GABAPENTIN 100 MG CAPSULE	GABAPENTIN
3	HALDOL 2MG TABLET	HALOPERIDOL
4	HYDROCODONE-APAP 5-325	HYDROCODONE BITARTE...
5	HYDROCODONE-APAP 10-325 TAB	HYDROCODONE BITARTE...
6	HYDROMORPHONE HCL 10 MG/ML	HYDROMORPHONE HCL/P
7	KETAMINE 100 MG/ML VIAL	KETAMINE HCL
8	LISINAPRIL 10 MG TABLET	LISINAPRIL
9	METHADONE 10 MG/ML ORAL CON	METHADONE HCL
10	METHADONE HCL 5mg TABLET	METHADONE HCL
11	MORPHINE 10 MG/ML VIAL	MORPHINE SULFATE
12	MORPHINE 4MG/ML VIAL	MORPHINE
13	PREDNISONE 1 MG TABLET	PREDNISONE
14	PSEUDOPHEDRINE 30MG TABLET	PSEUDOPHEDRINE HCL

Prev Create Selected Skip

Inventory Item

Scan Barcode

Drug Name

Secondary Name

Quantity DEA Class

Expires

Lot

Conversion Rate (This value declares how many units are in a single dispensing)

NDC

Par Level ☐ Use 11 Digit NDC

Max Qty ☐ Witness ☐ Security Code

Mfg Name

Mfg Expires

☐ Returnable

Save Cancel

If you would like more information or training for this new feature, please contact us at 1-855-633-2278 ext 2 or Support@phoenixltc.com.